


**VITEK 2™**  
 – technology **AST-GP75**

IVD

*Staphylococcus* spp., *Enterococcus* spp. and *S. agalactiae* Susceptibility
**SUMMARY AND EXPLANATION**

The VITEK® 2 Gram Positive Susceptibility Card is intended for use with the VITEK 2 Systems in clinical laboratories as an in vitro test to determine the susceptibility of ***Staphylococcus* spp., *Enterococcus* spp., and *S. agalactiae*** to antimicrobial agents when used as instructed in the Product Information manual.

**INSTRUCTIONS FOR USE**

A package insert is provided in the kit or downloadable from [www.biomerieux.com/techlib](http://www.biomerieux.com/techlib)

See the Product Information manual for additional Instructions for Use.

**STORAGE CONDITIONS**

Store at 2° to 8° C.

**CONTENTS OF THE CARD**

Antimicrobial	Code	Concentration §	Calling Range		FDA Indications for Use
			≤	≥	
Ampicillin <i>Enterococcus</i> spp. <i>S. agalactiae</i>	AM	0.5, 4, 8, 32	- 2 0.25	- 32 16	<i>Enterococcus</i> spp., <i>S. agalactiae</i>
Cefoxitin Screen	OVSF	6	NEG	POS	<i>Staphylococcus</i> spp.
Ciprofloxacin	CIP	1, 2, 4	0.5	8	<i>Staphylococcus</i> spp., <i>Enterococcus</i> spp.
Clindamycin	CM	0.06, 0.25, 1	0.125	4	MSSA**, MSSE**
Daptomycin <sup>NS</sup>	DAP	0.5, 1, 2, 4, 16	0.12	8	<i>S. aureus</i> , VSEfaeca**
Doxycycline	DO	0.25, 1, 2, 4	0.5	16	<i>S. aureus</i>
Erythromycin	E●	0.25, 0.5, 2	0.25	8	<i>Staphylococcus</i> spp., <i>Enterococcus</i> spp., <i>S. agalactiae</i>
Gentamicin	GM	8, 16, 64	0.5	16	<i>Staphylococcus</i> spp.
Gentamicin High Level (synergy)	HLS	500	S	R	<i>Enterococcus</i> spp.
Inducible Clindamycin Resistance	ICR●	CM 0.5, CM/E 0.25/0.5	NEG	POS	<i>Staphylococcus</i> spp.
Levofloxacin	LEV	0.25, 2, 8	0.12	8	<i>Staphylococcus</i> spp., <i>Enterococcus</i> spp., <i>S. agalactiae</i>
Linezolid	LNZ	0.5, 1, 2	0.5	8	<i>S. agalactiae</i> , <i>E. faecalis</i> , <i>E. faecium</i> , <i>S. aureus</i> , <i>S. epidermidis</i> , <i>S. haemolyticus</i>
Moxifloxacin	MXF	0.25, 2, 8	0.25	8	MSSA**
Nitrofurantoin	FT	16, 32, 64	16	512	<i>Staphylococcus</i> spp., <i>Enterococcus</i> spp.
Oxacillin	OX1	0.5, 1, 2	0.25	4	<i>Staphylococcus</i> spp.
Rifampicin	RA	0.25, 0.5, 2	0.5	32	<i>Staphylococcus</i> spp.
Streptomycin High Level (synergy)	HLS	1000	S	R	<i>Enterococcus</i> spp.
Tetracycline	TE	0.5, 1, 2	1	16	<i>Staphylococcus</i> spp., <i>Enterococcus</i> spp., <i>S. agalactiae</i>
Tigecycline <sup>NS</sup>	TGC	0.25, 0.5, 1	0.12	2	<i>E. faecalis</i> , <i>E. faecium</i> , <i>S. aureus</i> , <i>S. epidermidis</i> , <i>S. haemolyticus</i> , <i>S. agalactiae</i> , <i>E. casseliflavus</i>
Trimethoprim/Sulfamethoxazole <sup>c</sup>	SXT●	2/38, 8/152, 16/304	10 (0.5/9.5)	320 (16/304)	<i>S. aureus</i>
Vancomycin	VA●	1, 2, 4, 8, 16	0.5	32	<i>Enterococcus</i> spp., <i>Staphylococcus</i> spp., <i>S. agalactiae</i>

Numerical values are expressed in µg/ml.

§ Equivalent standard method concentration by efficacy.

NEG = Negative

POS = Positive

\*\*MSSA = Methicillin-susceptible *S. aureus*

\*\*MSSE = Methicillin-susceptible *S. epidermidis*

<sup>NS</sup> = The current absence of resistant isolates precludes defining any results other than susceptible. Isolates yielding MIC results suggestive of Nonsusceptible category should be submitted to a reference laboratory for further testing.

\*\*VSEfaeca = *E. faecalis* (vancomycin susceptible strains)

①, ② etc. = See performance characteristics identified by the drug code with this symbol in the Comment column in Systems Product Information.

<sup>c</sup> = Category agreement was established at the time of FDA clearance. Essential agreement was not established since test contains less than five discrete dilutions.

## QUALITY CONTROL

Antimicrobial	Code	CLSI® Quality Control Organisms VITEK 2 Results					
		<i>E. faecalis</i> ATCC® 29212™	<i>S. aureus</i> ATCC® 29213™	<i>E. faecalis</i> ATCC® 51299™	<i>S. aureus</i> ATCC® BAA- 1026™	<i>S. aureus</i> ATCC® BAA- 976™	<i>S. aureus</i> ATCC® BAA- 977™
Ampicillin <i>Enterococcus</i> spp. <i>S. agalactiae</i>	AM	≤2	0.5 – 2	-	-	-	-
Cefoxitin Screen	OVSF	-	NEG	-	POS	-	-
Ciprofloxacin	CIP	≤0.5 – 2	≤0.5	-	-	-	-
Clindamycin	CM	≥4	≤0.12 – 0.25* (*FDA/CLSI Broth Microdilution expected QC range = 0.06 – 0.25 µg/ml)	-	-	-	-
Daptomycin <sup>NS</sup>	DAP	1 – 4	0.25 – 1	-	-	-	-
Doxycycline	DO	2 – 8	≤0.5	-	-	-	-
Erythromycin	E	1 – 4	≤0.25 – 1	-	-	-	-
Gentamicin	GM	-	≤0.5 – 1	-	-	-	-
Gentamicin High Level (synergy)	HLG	S	-	R	-	-	-
Inducible Clindamycin Resistance	ICR	-	-	-	-	NEG	POS
Levofloxacin	LEV	0.25 – 2	≤0.12 – 0.5	-	-	-	-
Linezolid	LNZ	1 – 4	1 – 4	-	-	-	-
Moxifloxacin	MXF	≤0.25 – 0.5	≤0.25	-	-	-	-
Nitrofurantoin	FT	≤16	≤16 – 32	-	-	-	-
Oxacillin	OX1	-	≤0.25 – 0.5	-	-	-	-
Rifampicin	RA	-	≤0.5	-	-	-	-
Streptomycin High Level (synergy)	HLS	S	-	R	-	-	-
Tetracycline	TE	8 – ≥16	≤1	-	-	-	-
Tigecycline <sup>NS</sup>	TGC	≤0.12	≤0.12 – 0.25	-	-	-	-
Trimethoprim/Sulfamethoxazole <sup>c</sup>	SXT	-	≤10 (0.5/9.5)	-	-	-	-
Vancomycin	VA	1 – 4	≤0.5 – 2	-	-	-	-

Numerical values are expressed in µg/ml.

NEG = Negative

POS = Positive

<sup>NS</sup> = The current absence of resistant isolates precludes defining any results other than susceptible. Isolates yielding MIC results suggestive of Nonsusceptible category should be submitted to a reference laboratory for further testing.

<sup>c</sup> = Category agreement was established at the time of FDA clearance. Essential agreement was not established since test contains less than five discrete dilutions.

## LIMITATIONS

Perform an alternative method of testing prior to reporting of results for the following antibiotic/organism combination(s):

- Daptomycin: *Streptococcus agalactiae*
- Erythromycin: *Streptococcus agalactiae*

Perform an alternative method of testing prior to reporting of results when a Positive (+) result is obtained with the following antibiotic/organism combination(s):

- Cefoxitin Screen: *Staphylococcus saprophyticus*

The ability of the AST card to detect resistance with the following combination(s) is unknown because resistant strains were not available at the time of comparative testing:

- Ampicillin: *Streptococcus agalactiae*
- Linezolid: *Enterococcus* spp., *Staphylococcus* spp., *Streptococcus agalactiae*
- Tigecycline: *Enterococcus* spp., *Staphylococcus* spp., *Streptococcus* spp.

**NOTE:** A result for an antibiotic/organism combination which may have a limitation may be suppressed from reporting. Refer to the software user manual for instructions.

↓ MUST enter the following barcodes into "Flex Panel Entry" program before first use of this Susceptibility Card.



#### INDEX OF SYMBOLS

Symbol	Meaning
	Catalogue number
	In Vitro Diagnostic Medical Device
	Manufacturer
	Date of Manufacture
	Temperature limitation
	Use by
	Batch code
	Consult Instructions for Use
	Contains sufficient for <n> tests
	Authorized representative in the European Community

#### PATENTS

Product covered by one or more of U.S. Patent Nos. D414,272; D437,797; 5,609,828; 5,746,980; 5,804,437; 5,869,005; 5,932,177; 5,951,952; 6,267,929; 6,309,890 and 6,340,573; and Foreign Counterparts. Other Patents Pending.

#### WARRANTY

*bioMérieux disclaims all warranties, express or implied, including any implied warranties of MERCHANTABILITY AND FITNESS FOR A PARTICULAR USE. bioMérieux shall not be liable for any incidental or consequential damages. IN NO EVENT SHALL BIOMERIEUX'S LIABILITY TO CUSTOMER UNDER ANY CLAIM EXCEED A REFUND OF THE AMOUNT PAID TO BIOMERIEUX FOR THE PRODUCT OR SERVICE WHICH IS THE SUBJECT OF THE CLAIM.*

BIOMERIEUX, the blue logo, VITEK and bioLiaison are used, pending and/or registered trademarks belonging to bioMérieux SA or one of its subsidiaries, or one of its companies. The ATCC trademark and trade name and any and all ATCC catalog numbers are trademarks of the American Type Culture Collection. Any other name or trademark is the property of its respective owner.



**bioMérieux, Inc.**  
100 Rodolphe Street  
Durham, North Carolina  
27712 USA

**bioMérieux SA**  
Chemin de l'Orme  
RCS LYON 673 620 399  
69280 Marcy-l'Etoile / France  
Tel. 33 (0)4 78 87 20 00  
Fax 33 (0)4 78 87 20 90  
www.biomerieux.com

